


FEMORAL HEAD REMOVAL

with TREU 11-084 and 11-084-T

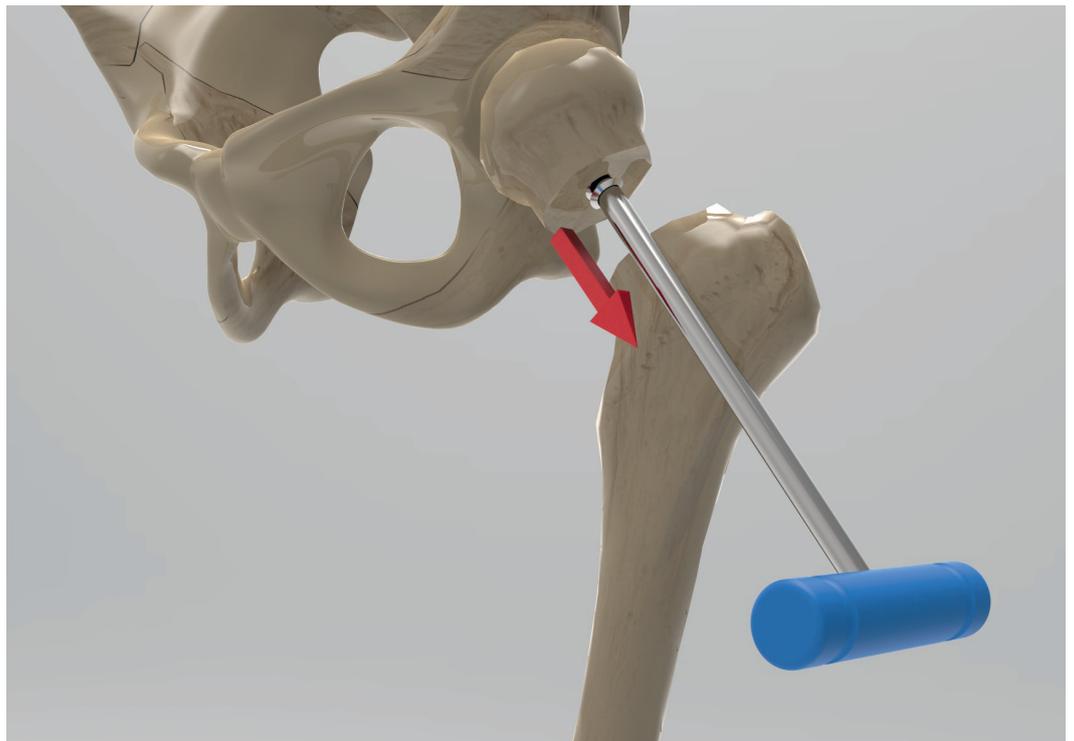

Start with an adequate exposure of the case area. Sufficient capsular incision is needed. Use a Hip Skid to dislocate the Femoral Head.
Cut through the femoral head with a power saw.

Use the 11-084 or 11-084-T and start screwing into the femoral head (as illustrated). The Screw should be inserted into the center of the head as deeply as possible, taking care to avoid protrusion beyond the articular surface of the femoral head. At least screw in about 6 complete rotations (appx. 25mm).
Divide the ligamentum teres as necessary.
Remove the Femoral Head by pulling out.
Exposure of the acetabulum is necessary. Remove bone fragments if needed.